



# DRUG ABUSE RAPID TESTS




- Results in 5-10 minutes
- Individual/Multi-panel screening
- Urine sample



- High sensitivity & Accuracy
- OEM combined test





# EASY TO READ

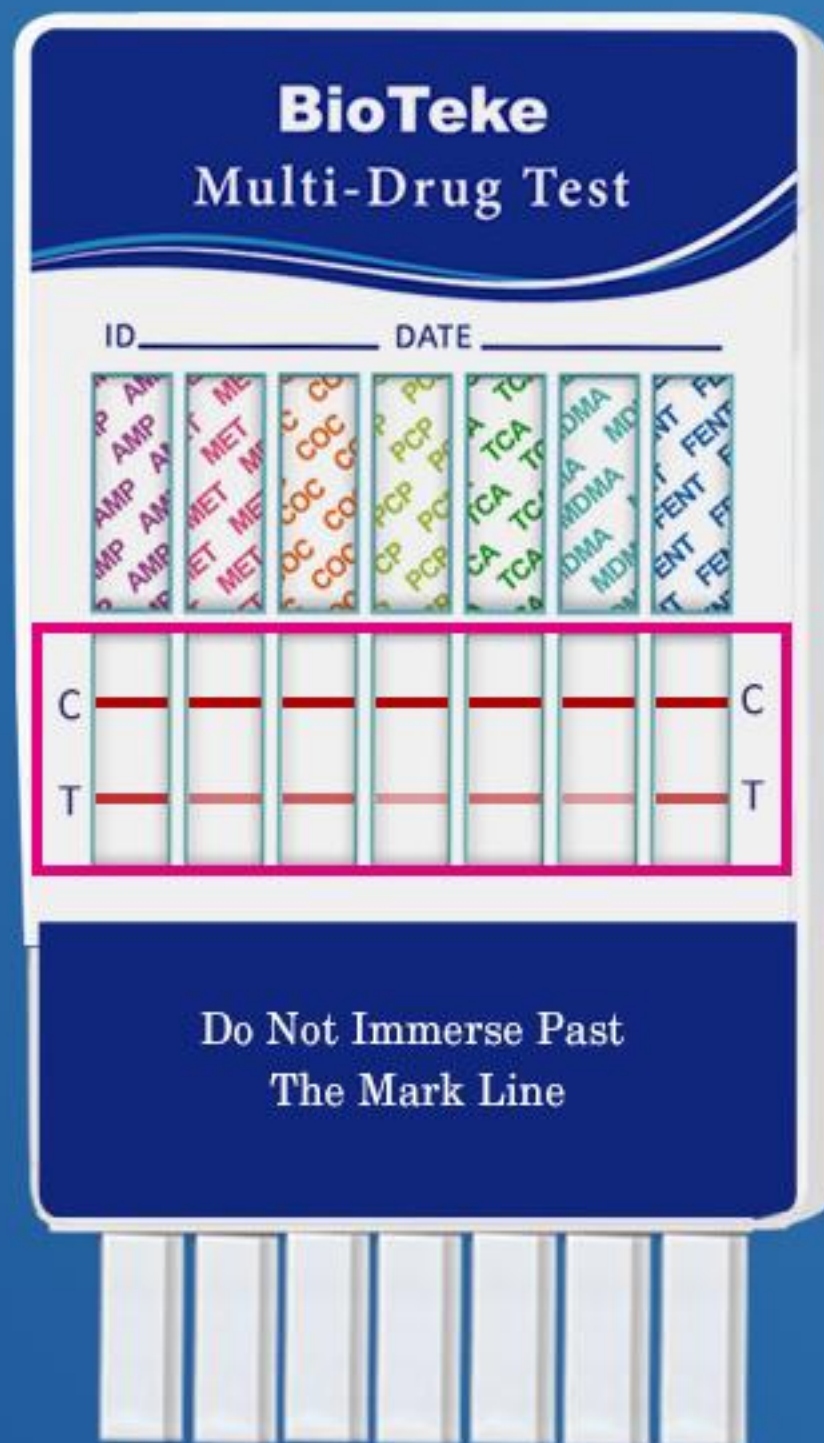


## NEGATIVE

A line shows up in both the Control (C) and Test (T) Regions.  
No matter how faint the Test (T) line is, result is negative.

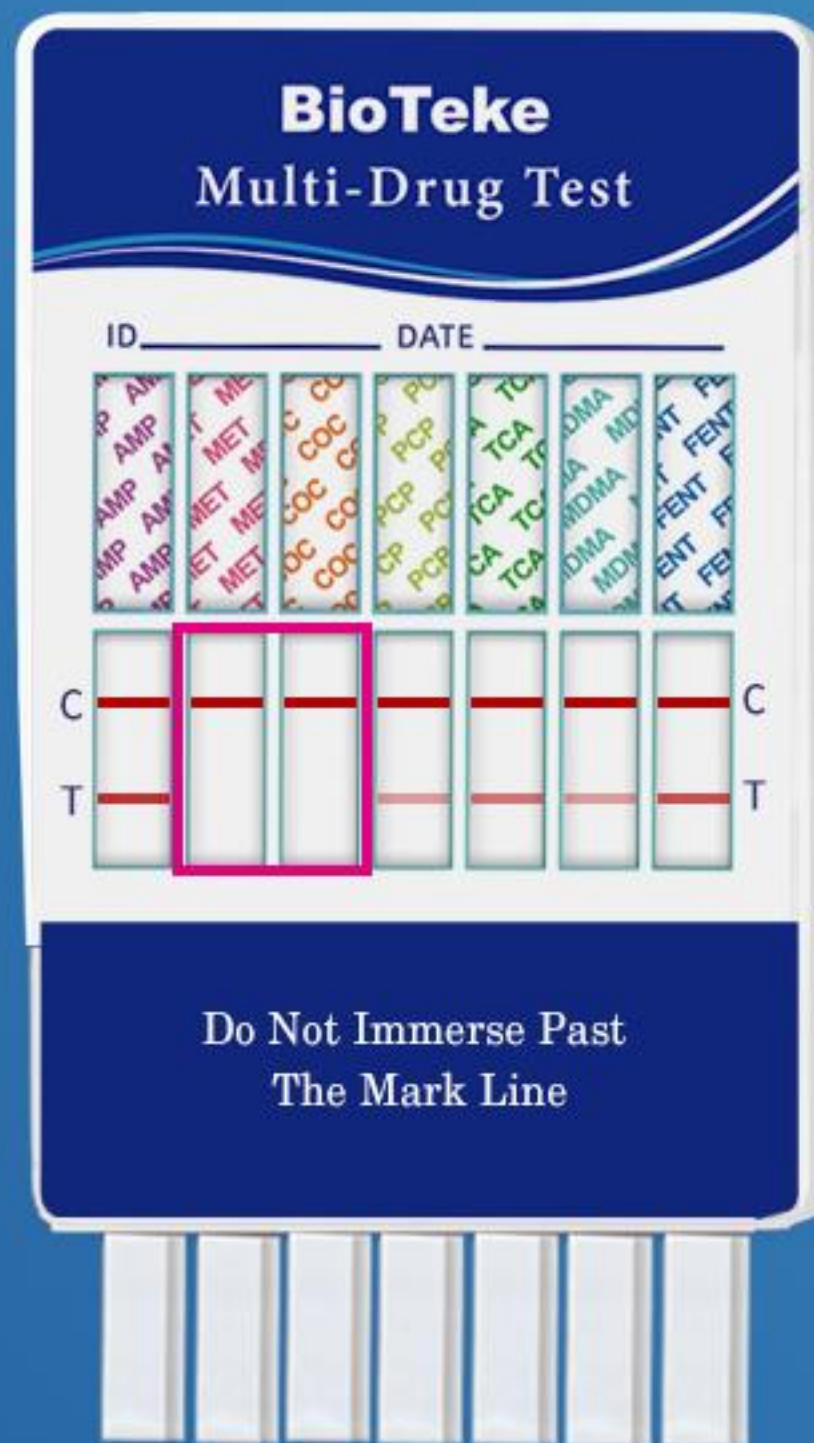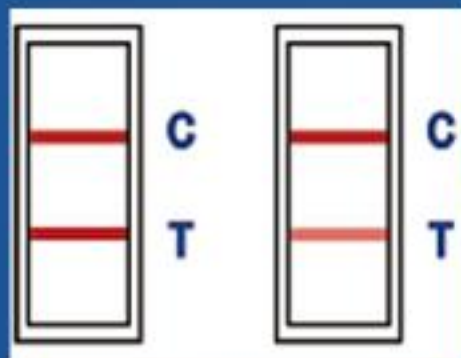


## PRELIMINARY POSITIVE

A line shows up in Control (C) region, No line shows up in Test (T) region.

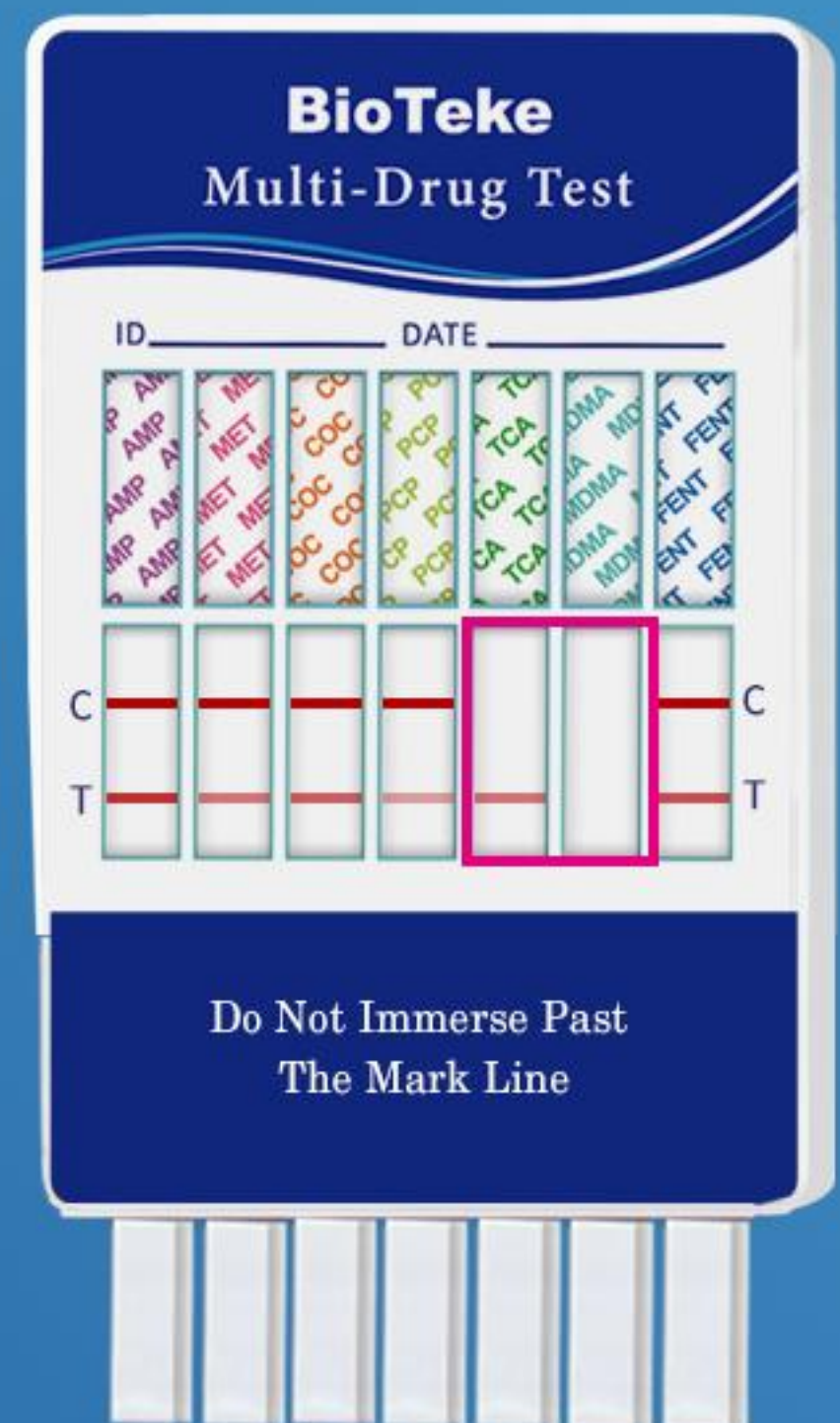

## INVALID

A line does not show up in either region, or only shows up in the Test (T) region.  
Make sure to retake test.

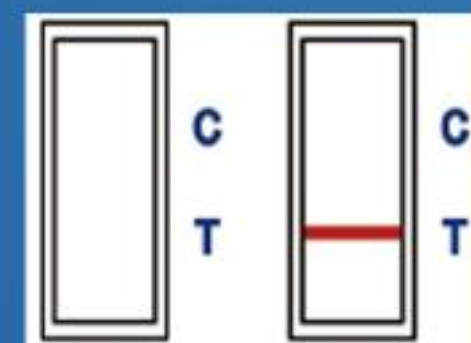



# FAST RESULT



1

## DIP

Submerge the test panel until all the dyes rise to the surface. Do not submerge past the "MAX" line.


2

## WAIT

Place the test on a dry, non-absorbent surface. Allow the test to sit for 5 minutes.



5-10  
minutes

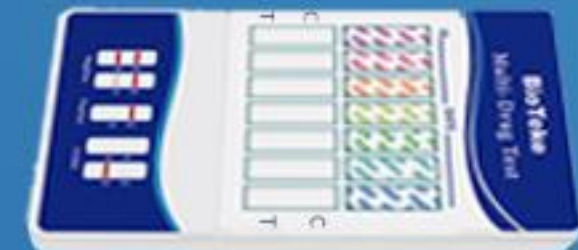

3


## EASY TO READ

Clear and readable results

4

## READ

Critical to read the results after 5 minutes for an accurate result.

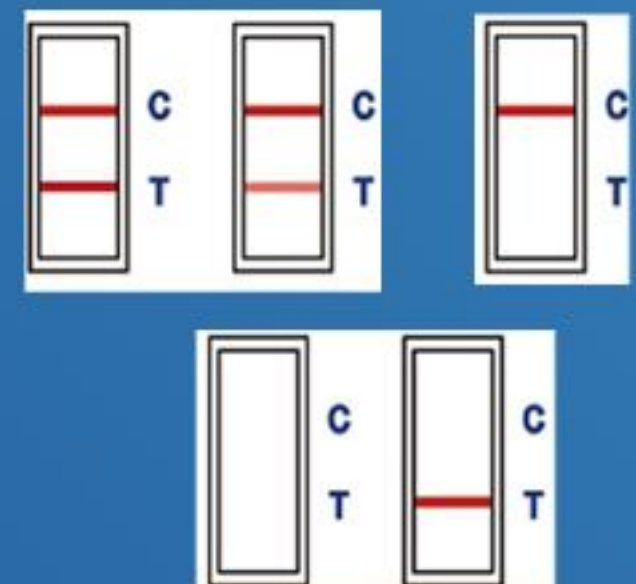




# CUT-OFF LEVEL & DETECTION TIME

SUBSTANCE TESTED	CUT-OFF LEVEL	MIN DETECTION TIME	MAX DETECTION TIME
FENT	1ng/mL	1-2 hours	1-3 days
THC	50ng/mL	2 hours	Up to 5+ days
MDMA	500ng/mL	2-7 hours	2-4 days
MOP	300ng/mL	2 hours	2-3 days
BZO	300ng/mL	2-7 hours	1-2 days
MTD	300ng/mL	3-8 hours	1-3 days
BAR	300ng/mL	2-4 hours	1-4 days
BUP	10ng/mL	1 hours	1-7 days
OXY	100ng/mL	2 hours	2 days
AMP	500ng/mL	2-7 hours	1-2 days
COC	150ng/mL	1-4 hours	2-4 days
MET	500ng/mL	2-7 hours	2-4 days
PCP	25ng/mL	4-6 hours	1-4 days
TCA	1000ng/mL	2 hours	7-10 days

WITH ANY POCT URINETEST, IT IS IMPORTANT TO NOTE THAT A NEGATIVE RESULT DOES NOT ALWAYS MEAN THE URINE SAMPLE IS FREE OF DRUGS. IT MAY CONTAIN A DRUG AT A CONCENTRATION THAT IS LOWER THAN THE CUT-OFF LEVEL.

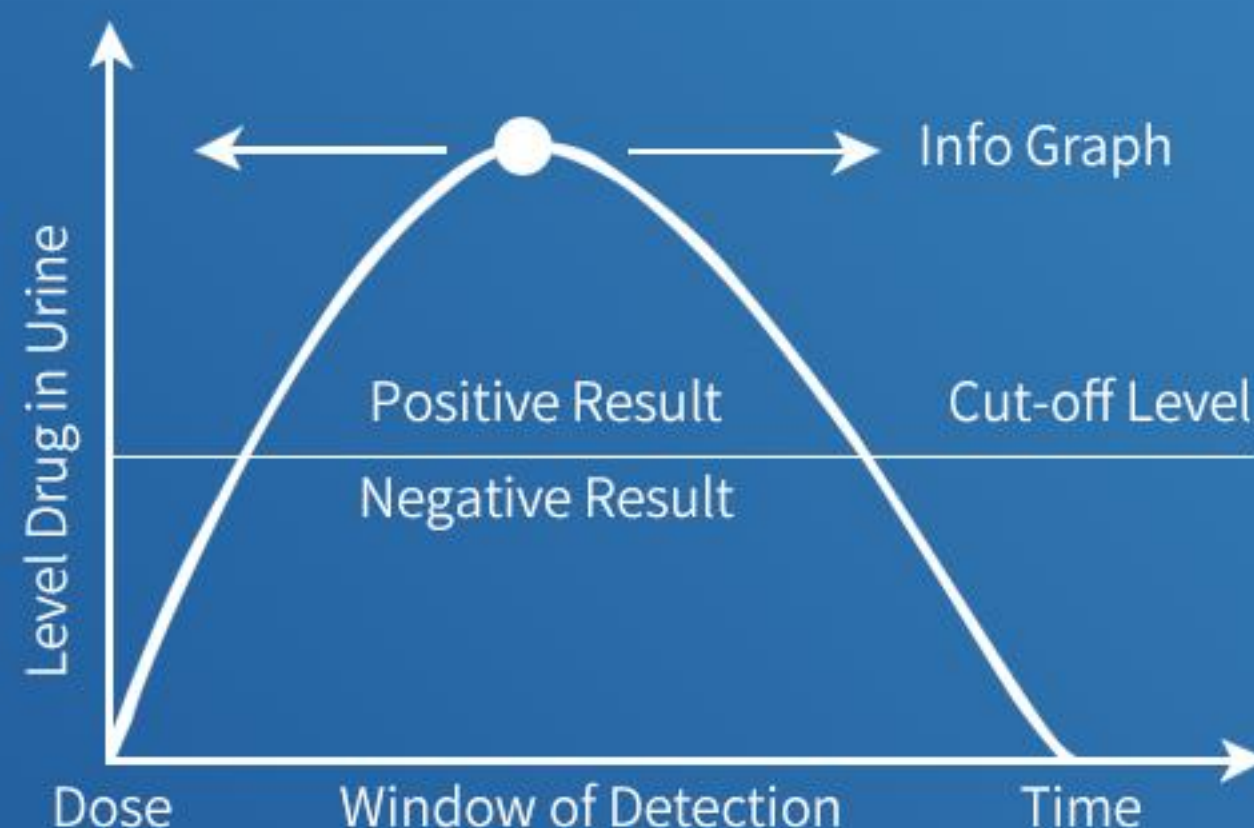



# DETECT 14 DRUGS IN ONE

- (FENT) FENTANYL
- (THC) MARIJUANA
- (MOP) MORPHINE
- (BZO) BENZODIAZEPINES
- (MTD) METHADONE
- (BAR) BARBITURATES
- (BUP) BUPRENORPHINE
- (OXY) OXYCODONE
- (AMP) AMPHETAMINE
- (COC) COCAINE
- (MET) METHAMPHETAMINE
- (PCP) PHENYL CYCLOHEXYL PIPERIDINE
- (TCA) TRICYCLIC ANTIDEPRESSANTS
- (MDMA) METHYLENEDIOXY METH-AMPHETAMINE

